



HIPGARD

The protective shields in HIPGARD are based on our specially patented horse-shoe-shaped foam shield. The shields are light-weight, shaped to fit the body's contour and feel very comfortable to wear. The pants hold a high content of elastane which ensures a perfect fit and also makes them suitable for fixation of an incontinence pad.

The patented shield technology of HIPGARD uses an energy shunting technique to reduce the fall impact by dispersing the energy of the fall away from the hip area and into the soft tissue. HIPGARD has been clinically tested on more than 7,000 users around the world and has proven to reduce hip fractures from falls.

Protector PE Foam

Seamless Briefs

67% Cotton
29% Polyester
4% Elastane



Care handling

HIPGARD can be washed at 60°C/140°F and tumble dried. Wash with similar colours and in a washing bag. Use of chlorine based bleach or chlorine based disinfection will decrease durability and function.

Please keep this instruction for further reference.

HIPGARD	Cm	Inch	Find your size
XS	65-85	26-34	
S	75-95	30-38	
M	90-110	36-44	
L	100-120	40-48	
XL	110-140	44-56	
XXL	120-150	48-60	

Sizes overlap. To find the user's correct size, measure the hips at the widest point. If the hip measurement lies between two sizes, choose the smaller size if the patient has slim thighs, and the larger size if thighs are wide or an incontinence pad will be worn.

Ramblegard Customer Service:

E-mail: sales@ramblegard.com
UK: +44 (0) 28 9448 3971



Certified according to ISO 9001, ISO 14001 and ISO 13485
Elmfield House, Rathenraw Industrial Estate, Greystone Road, Antrim, BT41 2SJ

Ramblegard
stay a step ahead



HIPGARD

The protective shields in HIPGARD are based on our specially patented horse-shoe-shaped foam shield. The shields are light-weight, shaped to fit the body's contour and feel very comfortable to wear. The pants hold a high content of elastane which ensures a perfect fit and also makes them suitable for fixation of an incontinence pad.

The patented shield technology of HIPGARD uses an energy shunting technique to reduce the fall impact by dispersing the energy of the fall away from the hip area and into the soft tissue. HIPGARD has been clinically tested on more than 7,000 users around the world and has proven to reduce hip fractures from falls.

Protector
PE Foam

Seamless Briefs
67% Cotton
29% Polyester
4% Elastane



Care handling

HIPGARD can be washed at 60°C/140°F and tumble dried. Wash with similar colours and in a washing bag. Use of chlorine based bleach or chlorine based disinfection will decrease durability and function.

Please keep this instruction for further reference.

HIPGARD	Cm	Inch	Find your size
XS	65-85	26-34	
S	75-95	30-38	
M	90-110	36-44	
L	100-120	40-48	
XL	110-140	44-56	
XXL	120-150	48-60	

Sizes overlap. To find the user's correct size, measure the hips at the widest point. If the hip measurement lies between two sizes, choose the smaller size if the patient has slim thighs, and the larger size if thighs are wide or an incontinence pad will be worn.

Ramblegard Customer Service:
E-mail: sales@ramblegard.com
UK: +44 (0) 28 9448 3971

OEKO-TEX®
CONFIDENCE IN TEXTILES
STANDARD 100
6234-7001 DTI
Testet for sundhedsfarlige stoffer:
www.oeko-tex.com/standard100



Certified according to ISO 9001, ISO 14001 and ISO 13485
Elmfield House, Rathenraw Industrial Estate, Greystone Road, Antrim, BT41 2SJ

Ramblegard®
stay a step ahead